


General-Use ConRad Thermal Blanket

Stop Heat Loss

How to Use the General Use ConRad® Thermal Blanket

Directions for Use – Prevent Phase 1 heat loss

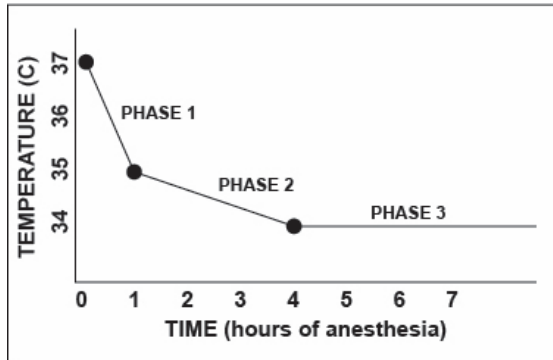

For General Use:

- Surgery
- Dental
- Endoscopy
- Pre-op
- Post surgical recovery
- X-ray
- CT Scan

- 1) Place the patient on a solid surface warming pad and covered with the ConRad Thermal Blanket, in the pre-oxygenation phase, prior to the induction of anesthesia.
- 2) Keep the patient on the warming pad and covered with the ConRad Thermal Blanket as much as possible while prepping and applying patient monitors.
- 3) Keep the patient covered with the ConRad Thermal Blanket when moving the patient to the procedure table.
- 4) Make certain that an insulating layer, such as a regular blanket or another ConRad Thermal Blanket, is underneath and between the patient and the underlying surface.
- 5) Once positioned, make certain that the ConRad Thermal Blanket drapes around the patient as close as possible.
- 6) Place the ConRad Thermal Blanket so that the side labeled PATIENT SIDE is next to the patient.
- 7) Clean with soap and water or a germicidal solution.
- 8) The ConRad Thermal Blankets may be used with other patient warming systems such as forced air, electrical and water warming systems.

Not For MRI Use